**Ganter**  
natural walking

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T +43 7719 8811700 F +43 7719 8811795 E office@ganter-shoes.com

www.ganter-shoes.com FN 230441i Landesgericht Ried/I. UID-Nr. ATU56331188

# Instructions for Use

**Please read these instructions carefully before using this product. Follow all instructions, especially the safety instructions.**

## **PRODUCT DESCRIPTION**

Off-the-shelf protective footwear for people with diabetes protects sensitive feet from pressure points and injuries. These shoes are designed to relieve the foot as much as possible and offer sufficient room for orthopaedic devices.

## **INTENDED PURPOSE / INDICATIONS**

Protective footwear is suitable for people with diabetic foot syndrome.

- Neuropathy or polyneuropathy
- High-risk feet after ulcer healing
- Partial amputation
- Hallux rigidus
- Changes to the foot caused by rheumatism
- Other mechanical irritations that lead to localised pressure points

Custom orthopaedic shoes are not yet required due to the physiological condition of the foot.

## **USE-RELATED RISKS AND CONTRAINDICATIONS**

Please consult your doctor before using this product in the following cases:

- Skin disorders or injuries to the foot
- Signs of infection or inflammation such as redness, heat or swelling
- Circulation problems
- Lymphatic drainage problems
- Sensory issues in the feet.
- In cases of a contracted foot misalignment or partial amputation proximal to the metatarsal joints, a custom shoe is typically necessary.
- In the case of diabetic-neuropathic osteoarthropathy, a relief orthosis or a full contact plaster cast is necessary due to the remodelling of the bones of the foot.
- If the foot is infected, a wound-care or off-loading shoe must be worn.

**Please contact your doctor immediately if you notice any unusual changes.**

## **NOTES ON USE**

Before first use, the protective shoes must be adjusted to the specific needs of the wearer by a trained professional. This professional is responsible for the proper fit and function. If the shoe is not adjusted to meet the wearer's needs, product liability is not guaranteed. The correct way to put on the shoe is described below:

1. Put your foot into the open, untied/unbuckled shoe. It would be the best to use a shoehorn to put on the shoe.
2. Make sure that the foot is positioned securely/correctly in the shoe, i.e. the heel is positioned as far back as possible in the shoe.

3. Depending on the model, fasten the shoe with the Velcro straps, zipper or laces. The foot must be well secured in place, and the shoe should not be too loose or too tight.
4. Do not fasten the shoe too tightly to avoid creating pressure points and circulation problems.
5. Depending on the model, Velcro straps can be shortened as needed.
6. Check the shoes for foreign objects before putting them on.
7. Never walk barefoot if you have neuropathy (impaired sensation) in the feet.
8. When breaking in your new shoes, you should check your feet for sores regularly.

**If you experience problems with your feet or shoes, please contact your doctor or orthopaedic specialist immediately.**

## SAFETY INSTRUCTIONS AND RESIDUAL RISKS

-  Fitting of and adjustments to the protective footwear, as well as an introduction to the product, must be carried out only by a trained professional.
-  The product is intended to be used by one person only.
-  If you experience excessive pain, swelling, numbness or sensitivity, unusual reactions or an uncomfortable feeling while wearing, please contact your doctor or orthopaedic technician immediately.
-  The product is not to be worn in water and must be protected against heat exceeding 50°C. Direct sunlight may lead to discolouration. Changes in colour have no effect on the functionality of the footwear.
-  If the product is damaged, please replace it immediately or have it repaired by a qualified specialist.
-  Depending on the country, GANTER protective footwear for people with diabetes may be considered prescription products that should be worn only under medical supervision.

## CLEANING AND CARE

To extend the life of the product, dirt and debris should be removed with a soft, damp brush after each use. Do not use detergents or solvents. Shoes should never be washed in the washing machine. Air dry your shoes at room temperature in a place with good ventilation, protected from direct sunlight and heat. Your shoes will dry faster when stuffed with newspaper. Let shoes air out by removing the footbed of the shoe. Sand and seawater can damage your shoes. Clean your shoes thoroughly after exposure to such conditions.

**Leather:** Leather is a natural product. In order to prevent leather from ageing, we recommend treating shoes with leather-care products. Please note that such treatment may change the appearance of your shoes. It will not, however, affect the shoes' function or properties.

**Lining:** We recommend regularly cleaning shoe linings and always wearing socks. The SENSITIV lining can easily be cleaned with a damp cloth or sponge.

**Laces:** Laces are everyday items that may wear down. They should be replaced when necessary to ensure the product functions as intended.

**Velcro fastener:** The functionality of a Velcro fastener is only guaranteed when it is kept clean. Clean as needed.

## GENERAL PROVISIONS

- Due to the indication-based selection, as well as customised adjustments and hygienic reasons, protective shoes should only be worn by one person. These protective shoes are not suitable for reuse or multiple uses.
- Check the product for wear and damage every 4 weeks. The properties of the protective shoes deteriorate with repeated use (wearing). For this reason, it is not possible to specify a definitive service life or period of wear. The shoes should be replaced when worn.
- These protective shoes may be worn in combination with other medical devices. However, a combination is only permitted with diabetes-adapted insoles that have been selected and adjusted by a trained professional.

## TECHNICAL INFORMATION / PARAMETERS

Materials may vary depending on model:

- Full grain leather, nubuck, suede
- Textiles made of polyester, fleece made of nylon/polyurethane
- EVA (ethylene and vinyl acetate copolymer)
- PU (polyurethane)
- PVC (polyvinyl chloride)
- Glass fiber reinforced polyamide
- Latex foam
- Cork
- Rubber
- Polyamide (Velcro fastener)
- Metal (eyelets, zippers)

## DISPOSAL

The product and its packaging must be disposed of in accordance with legal provisions.

## WARRANTY

Current valid legal provisions apply. Failure to follow the instructions for use, improper cleaning and technical changes and additions to the product carried out by non-professionals will invalidate the warranty and the product liability in general.

The CE mark confirms compliance with the relevant requirements of the Medical Device Regulation  
Date/Amended version 2020-02

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GANTER Shoes GmbH A-4775 Taufkirchen/Pram Joseph-Lorenz-Straße 2